

Maintenance and cleaning instructions

Mechanisms

All of recliners and sleepers employ maintenance free mechanisms with self-lubricating polymer bushings. Removal of any dust and debris buildup is recommended every three months.

Wood/Thermofoil Finishes

Use a hospital-grade combined disinfectant/detergent but make sure to dry promptly. DO NOT use steam cleaners, pressure washers, or abrasive cleaners on any chairs with wood components.

Arm caps (Solid Surface/Polyester)

Tested with the following cleaners:

- 1 - PDI Sani-Cloth plus - Quat based disinfectant
- 2 - Dispatch - bleach based disinfectant - 1:10 dilution
- 3 - Virex Tb - Quat based disinfectant

DO NOT use abrasive cleansers.

Arm caps (Upholstered)

These arm caps are upholstered please refer to the supplier cleaning specifications

Powder-coated Finishes

Tested with the following cleaners:

- 1 - PDI Sani-Cloth plus - Quat based disinfectant
- 2 - Dispatch - bleach based disinfectant - 1:10 dilution
- 3 - Virex Tb - Quat based disinfectant

DO NOT use abrasive cleansers.

Upholstery

Obtain the fabric care specifications from the upholstery supplier.